

Introduction

Gonorrhea is a sexually transmitted infection caused by the bacterium *Neisseria gonorrhoeae*. The infection is popularly known as the clap. Gonorrhea (the clap) is one of the most common infectious bacterial diseases and is most frequently transmitted during sexual intercourse, including vaginal, oral and anal sex. The causative organism can infect the throat, producing a severe sore throat. It can also affect the anus and rectum, producing a condition called proctitis. Even though most people who are infected have no symptoms, they may include a painful or burning sensation when urinating, vaginal bleeding between periods, increased vaginal secretion, often green yellow in colour, itching and burning of the vagina, urge to urinate in women and discharge from the penis, burning with urination, frequently urinating small amounts and pain/swelling of one or both testicles in men. The only prevention is by not having sex, using condoms, or having sex with only one partner who is not infected. For sexually active people, screening is recommended at least once a year. After a discreet home test that is the same for women and men, a Gonorrhea infection can be easily cured, usually with antibiotics.

Important

- ❖ Before using the home test, read these instructions carefully.
- ❖ This test is for single use only. Therefore, it is very important that you read all the instructions and understand what you should do before you start the test. Perform all instructions accurately as described in the instructions. Not performing all steps properly can lead to an incorrect result.
- ❖ The kit should be stored at 2-30°C until the expiry date printed on the sealed pouch.
- ❖ The test cassette must remain in the sealed pouch until use.
- ❖ Not for internal use. Avoid contact between hands and eyes or nose when performing the test.
- ❖ Do not eat, drink or smoke in the vicinity of the test.
- ❖ The test should not be used after the expiration date, indicated on the package or when the packaging has been damaged or opened.
- ❖ Do not reuse tests.
- ❖ Avoid cross-contamination of specimens by using a new extraction tube for each specimen obtained.
- ❖ Do not interchange or mix reagents from different lots. Do not mix solution bottle caps.
- ❖ Do not use cytology brushes if pregnant.

Note:* If any symptoms persist, regardless of the outcome of the test, we advise you to contact your GP.

If you have any other questions. Don't be shy to email us @ info@yourselftests.com

Performing the test


Not following the instructions can lead to an incorrect result. Read all steps carefully.

Step 1 Wash your hands thoroughly with soap and warm water, rinse and dry them with a clean towel.

Step 2 If you are doing the test now, open all packaging. Place all test materials on an available clean, flat surface.

Step 3 Women: open the vagina with two fingers and put the sterile cotton swab into the vaginal opening until the tip is no longer visible. If there is visible discharge, remove this with the cotton swab first. (shown on 1st picture)

Men: put the sterile cotton swab 2 to 4 cm in the urethra. (shown on 2nd picture)

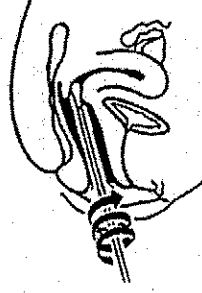

1.



2.

Step 4 Turn the sterile swab around 5 to 10 seconds.

Step 5 Place the swab in the enclosed test tube.

Step 6 Add 8 drops of the Buffer 1 to the test tube.

Step 7 Simultaneously squeeze the test tube 15 times and turn the cotton swab around.

Step 8 Hold the cotton swab in the test tube for 2 minutes.

Step 9 Next, add 8 drops of the Buffer 2 to the test tube.

Step 10 Again, simultaneously squeeze the test tube 15 times and turn the cotton swab around.


Step 11 Remove the cotton swab and squeeze out the contents into test tube. Close off the test tube with the enclosed cap.

Step 12 Add 3 drops of extracted specimen from the extraction tube to the specimen well (S) on the test cassette.

You can read off the test results after 10-15 minutes (do not read off the results later) if any symptoms persist, regardless of the outcome of the test, we advise you to contact your GP.

Interpretation of results

During the 15-minute waiting time 1, 2 or no coloured lines will appear in the oval compartment of the test cassette.

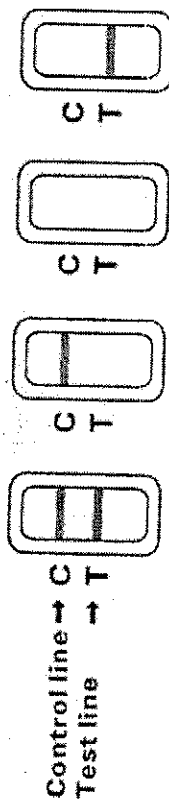

Positive Negative Invalid Invalid

Positive (+) - Two coloured bands appear at "C" and "T". This confirms the presence of gonorrhea. Please visit a doctor or licenced medical facility for further testing.

Negative (-) - Only one coloured band appears at "C". This confirms no presence of gonorrhea. To confirm you are not infected with Gonorrhea, a second test is advised.

Invalid - No visible band at all, or there is a visible band only at "T", but not at "C". Repeat with a new test kit.

Do not use the self-test again. You can order new one on www.yourselftests.com

Product manufactured by:
Hangzhou Alltest Biotech Co., Ltd.

Product distributed by:

Lintor Ltd.
27 Gen. Kiselov str., Varna, 9000
Bulgaria

E: info@yourselftests.com
W: www.yourselftests.com

Gonorrhea test

Safe and discreet testing at home

Instructions

Important: Before using the home test, read these instructions carefully.

Kit components provided:

- ❖ Individually packed test cassette
- ❖ Sterilized swab
- ❖ Buffers (Reagent 1, Reagent 2)
- ❖ Plastic extraction tube with a cap

Note: Also needed for this test: A clock, alarm clock or stopwatch to keep an eye on the time.

We will be happy to help!

If you experience any difficulty or need any help, please feel free to contact us at info@yourselftests.com